Matopat W Individual dressing

DESCRIPTION

The individual dressing consists of two absorbent non-woven pads (one of the pads is permanently attached, while the other can be moved along the bandage, which allows for dressing various types of wounds, e.g., through-and-through gunshot wounds), an elastic woven bandage and a loose sheet of polyurethane film.

INTENDED USE AND TARGET GROUP

Intended use:

- stopping severe bleeding: applying pressure directly to the wound to control and stop bleeding;
- treating injuries: dressing gunshot wounds (including through-and-through wounds) and other wounds characterised by significant blood loss;
- as an occlusive dressing: a sheet of polyurethane film is used to immediately seal (occlude) open chest wounds, minimizing the risk of tension pneumothorax.

Target group (users):

The dressing is intended for both professional users and users trained in tactical and medical first aid:

- Medical personnel;
- Armed forces;
- Uniformed and law enforcement services.

STERILITY

Sterile, radiation-sterilized device.

PREPARATION FOR USE AND METHOD OF USE

I. Dressing preparation and checking

- Before opening, check that the packaging is intact. Damaged packaging means loss of sterility;
- Locate the incisions and tear open the packaging, remove the dressing;
- Do not use a damaged or visibly contaminated dressing;
- Avoid touching the part of the compress to be placed directly on the wound.

II. Application of compresses

- Single wound/bleeding - fold both compresses together. Apply directly to the wound with maximum pressure;
- Through-and-through wound - place the fixed compress on the entry wound, move the movable compress and place it on the exit wound.

III. Bandaging

- While holding the compresses on the wound, wrap an elastic bandage around the injured part of the body;
- Secure the end of the bandage with a clip.

IV. Use of polyurethane film

- In the case of a chest wound with suspected tension pneumothorax, the film serves as an occlusive dressing (seal);
- Polyurethane film for chest wound sealing is intended for professional users only;
- Apply the film directly to the wound;
- Secure the film with a bandage on three sides (e.g., top and sides, depending on the position of the victim) so that the free edge acts as a vent.

CONTRAINDICATIONS

Do not use the device for any purpose other than those specified in these instructions.

SAFETY PRECAUTIONS

1. The device should be used in a sterile condition only;
2. Opening the individual packaging of the device is equivalent to its use. A sterile device cannot be re-sterilized;
3. Single-use only device;
4. Reuse of the device involves a very high risk of patient infection and is unacceptable;
5. The device retains its sterility within the specified shelf life, only in its original packaging and stored in accordance with the storage conditions specified on the packaging;
6. The device should be used only in the form and size in which it is supplied.

STORAGE

Storage and transport conditions in accordance with the information provided on the packaging.

DISPOSAL

Dispose of as medical waste in accordance with applicable local regulations.

REPORTING SERIOUS MEDICAL INCIDENTS

Report serious medical incidents to: qualitymanagement@tzm-global.com and the relevant local authorities.

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Toruńskie Zakłady Materiałów Opatrunkowych S.A.